Manual

Elma 609

Dansk/Norsk	3 - 9
Svenska	10 -16
English	17- 23

EAN: 5706445840526

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English

Elma 609 User Manual

Please read this manual before switching the unit on.
Important safety information inside

1. General Description

Elma 609 is specially designed to take the body temperature of a person regardless of room temperature. Depending on various skin types and thickness, there may be temperature difference.

2. Safety Information

- This device must only be used for the purposes described in this instruction manual
- This device must only be used in an ambient temperature range between 10 and 40°C
- Do not expose this thermometer to electric shocks.
- Do not expose this thermometer to extreme temperature conditions of > 50°C or < 0°C
- Do not use the device in relative humidity higher than 85%.
- Do not use the device near large electromagnetic fields like cordless or cell phones.
- Keep the device away from water and heat, including direct sunlight.
- Do not drop or knock the device, and do not use if damaged.
- It may affect the accuracy of measurements when the forehead is covered by hair, perspiration, cap scarf or other factors. (See Part 10-5)
- Please take the temperature behind the ear lobe. (See Part 10-5)
- Keep the measuring distance as 5cm-15cm (2in-5.9in). (See Part 10-4)
- **Elma 609** must be acclimated 15 to 20 minutes before use.
- Clean the glass with a cotton wool, lightly moistened with 70% alcohol.

Importance:

- Before taking the temperature make sure to remove hair and perspiration from the forehead.
- Selecting "**Body mode**" to measure the body temperature.
- Selecting "**Surface mode**" to measure the surface temperature.
- Use of this thermometer is not intended as a substitute for consultation with your physician.
- Should a problem occur with your device, please contact your retailer.
Do not attempt to repair the device yourself.
- According to EMC standard, the medical electronic products should be maintained specially.

3. Features

- Precise non-contact measurements
- User selectable °C or °F
- Selectable Body and Surface temp
- Set Alarm value
- Memorization of the last 32 measurements
- Automatic Data Hold & Auto power off
- Automatic selection range and Display Resolution 0.1°C (0.1°F)
- Backlight LCD display

4. Intended use

Elma 609 is designed for body surface and forehead temperature measurement for infants and adults without contact to human body.

Elma 609 can also be used to measure the temperature of a baby-bottle or bath, or room temperature (by using the **Surface Temp** function).

Normal Temperatures according to measurement method

Measurement method	Normal temp °C	Normal temp °F
Rectal	36,6 - 38,0°C	97,9 - 100,4°F
Oral	35,5 - 37,5°C	95,9 - 99,5°F
Axillary	34,7 - 37,3°C	94,5 - 99,1°F
Ear	35,8 - 38,0°C	96,4 - 100,4°F


The temperature of the human body varies throughout the day. It can also be influenced by numerous external factors: age, sex, type and thickness of skin.

Normal temperatures according to age

Age	Temp °C	Temp °F
0-2 years	36,4 - 38,0°C	97,5 - 100,4°F
3-10 years	36,1 - 37,8°C	97,0 - 100,0°F
11-65 years	35,9 - 37,6°C	96,6 - 99,68°F
> 65 years	35,8 - 37,5°C	96,4 - 99,5°F


5. Configuration

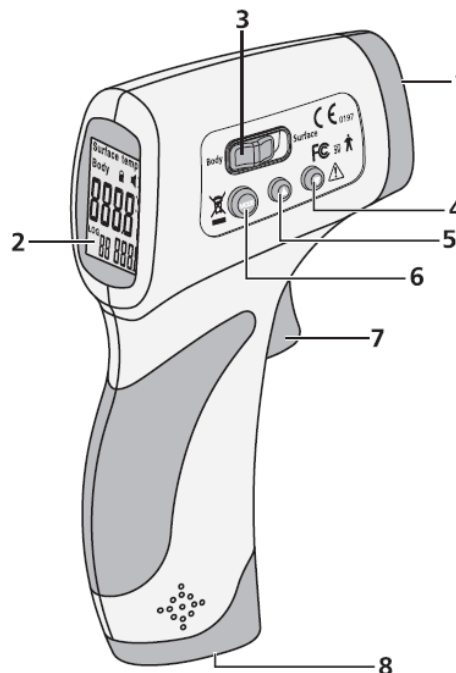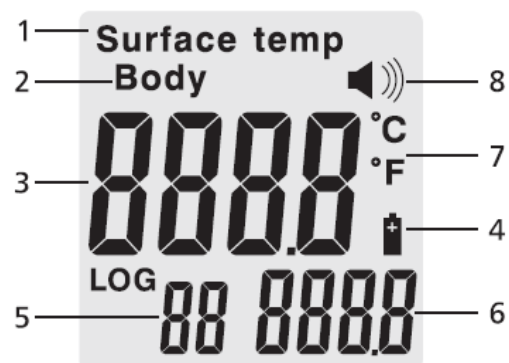1. IR Sensor
2. LCD Display
3. Mode Selection
4. Down Button
5. Up Button
6. Mode Button
7. Measurement Trigger
8. Battery Cover

6. Indicator

1. Surface mode symbol
2. Body mode symbol
3. Digital readout
4. Battery symbol
5. The order numbers
6. Saved data measurements
7. Temperature scale °C (Celsius)/ °F (Fahrenheit)
8. Buzzer symbol

7. Description of symbols

	The device is in accordance with Medical Device Directive 93/42/ EEC
	The device is in accordance with FCC Part 15 Subpart B: 2007 /Radio Frequency Devices IC Regulation ICEC-00 3: 2004 Interference-causing Equipment Standard-Digital Apparatus
	3V DC power supply
	Type B equipment
	In order to protect the environment, please recycle the battery according to the local regulations
Body Surface	Indication of mode
	Attention, consult accompanying documents

8. Technical Specifications

Normal Conditions of Use

Display Resolution	0.1°C (0.1°F)
Operating Temperature	10 to 40 °C (50 to 104°F)
Storage Temperature	0 to 50°C (32 to 122°F)
Humidity Rate	≤ 85%
Power	DC 3V (2 x AA batteries)
Size	149 x 77x 43 mm / 5.9 x 3 x 1.7 in (L x W x H)
Weight	Gross 400g / Net 172g

Measuring Range

In Body Mode	32.0 to 42.5°C (90 to 108.5°F)
In Surface Mode	0 to 60°C (32 to 140°F)
Accuracy	± 0.3°C (0.54°F)
Measuring Distance	5 cm - 15 cm (2 in - 5.9 in)
Automatic Stop	7 sec.

Elma 609 Precision

32 - 35,9°C (89,6 - 96,6°F)	± 0.3°C / 0.5°F	According to ASTM Standard
36 - 39°C (96,8 - 102,2°F)	± 0.2°C / 0.4°F	
39 - 42,5°C (102,2 - 108,5°F)	± 0.3°C / 0.5°F	E1965-1998 (2003)

9. Calibration temperature difference

For stable and reliable results, the thermometer provides a user-correction function, the operation of the method is as follows:

1st step: Take the temperature of a person using a conventional thermometer, you will get 37.5°C (99.5°F) for instance.

2nd step: Take the temperature of the same person using the **Elma 609** keeping the 5 to 15 cm (2 in to 5.9 in) distance between the thermometer and the forehead (Take care to remove any obstacle which could alter the measurement (hair, perspiration ...)). If you get 37.5°C (99.5 °F), the **Elma 609** is properly set and ready for use.

If you get a lower temperature, 36.4°C (97.5°F) for example, your difference is 1.1°C (2°F). You should adjust the temperature on the **Elma 609** and add the difference, i.e. 1.1°C (2°F).

To do it, press the "**MODE**" button for 2 seconds, the screen displays **F1**, press "**MODE**" button again until you get **F3**. Press "**UP**" button in order to add the difference (in our ex. 1.1°C (2°F))

3rd step: To check, take the temperature again using the **Elma 609**.

10. Use

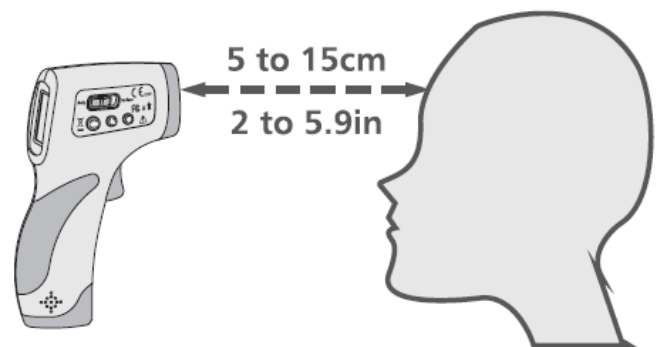
10.2 Install battery

10.3 For the first use or when inserting new batteries wait between 10 minutes for the warm-up of the apparatus and when inserting the new batteries.

If the device is not used for a long time, the device will test the room temperature first and will delay turning on for one or two seconds.


10.3 Aim towards the forehead, from a distance of 5cm (2in), press the measuring key, the temperature is displayed immediately.

Make sure there is no hair, perspiration, cosmetic or cap covered on the forehead.

When the room temperature is significant difference, or there is perspiration on the forehead, you can take the temperature behind the ear lobe.

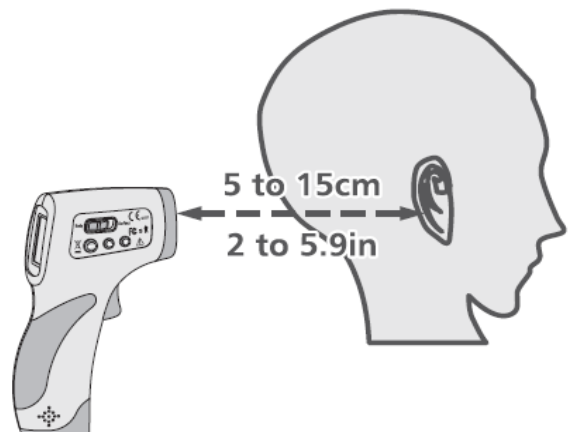Making sure there is no hair, perspiration, cosmetic or cap covered.

11. Measuring Operation

11.1 Choosing the temperature Unit - F1 Function

Press "**MODE**" button for 2 sec., the screen displays: **F1**.

Select "**DOWN**" for degrees Celsius °C, "**UP**" for degrees Fahrenheit °F.

11.2 Alarm Setup - F2 Menu

Press "**MODE**" button for 2 sec., the screen displays: **F1**. Press twice the "**MODE**" button to get **F2**. Select "**UP**" to increase the threshold by 0.1°C (0.1°F), "**DOWN**" to reduce it by 0.1°C (0.1°F).

Note: The alarm threshold default value is 38°C (100.4°F).

11.3 Calibration temperature difference - F3 Menu

To adjust the total variation of your **Elma 609**. Press the "**MODE**" button for 2 sec., the screen displays: **F1**. Press "**MODE**" button two times to get **F3**. Select "**UP**" to increase the difference by 0.1°C (0.1 °F), "**DOWN**" to reduce it by 0.1°C (0.1 °F). In cases of seasonal or environmental changes a verification and adjustment should be carried out.

Note: This function is only effective in **Body mode**

11.4 Buzzer ON/OFF - F4 Menu

Press the "**MODE**" button for 2 sec. the screen displays: **F1**. Press three times to get **F4**.

Select "**UP**" to activate the buzzer (a sound icon  is displayed on the LCD screen), press "**DOWN**" to stop it (the icon disappear).

11.5 Cal. ON / OFF- F5 Menu

Must always be set to **OFF** - otherwise not used.

11.6 Exiting the Setting Mode

Press "**MODE**" button until the screen turns off.

11.7 The Elma 609 is specially designed to take the body temperature of a human being.

For this, use the **Body mode**. Measurement range for **Body mode**: 32 to 42.5°C (89,6 to 108,5°F)

You can also use the **Elma 609** to measure the temperature of an area or an object, food, liquid or the room temperature. For this, use the **Surface mode**. Measurement range for **Surface mode**: 0 to 60°C (32 to 140°F).

Important: The area temperature differs from the internal body temperature.

To obtain the internal temperature always use the **Body mode**.

Please make sure to select the **Body mode** for an internal temperature reading and the **Surface mode** for an external area reading (bottle, bath, room...).

11.8 Data Memory

Data memories automatically after measurements, which will be displayed at the right corner of the LCD. Press "**UP**" or "**DOWN**" button to display the last temperature measurement.

Under the condition of **power off**, press "**UP**" and "**DOWN**" button together for two sec., to display the last measurement. At this mode if change the order number to **0** and press the "**MODE**" button, to delete all memory data.

11.9 Changing the batteries

Display: When the LCD displays , the battery is used.

Operation: Open the lid and change the batteries, taking care with the correct positioning. A mistake could cause damage to the apparatus and compromise the guarantee of your

Elma 609. Never use rechargeable batteries. Remove the battery if it is not required for extended periods in order to avoid damage to the thermometer resulting from a leaking battery.

11.10 Durability in use

The **Elma 609** was conceived for an intense and professional use, its durability is guaranteed for 40.000 measurements.

12 Advice

- The protective glass over the lens is the most important and fragile part of the thermometer, please take great care of it.
- Do not recharge non rechargeable batteries, do not throw in fire.
- Do not expose the thermometer to sunlight or water.

13 Maintenance and Cleaning

- The Infrared Sensor is the most precise part, and it must be protected carefully.
- Clean the device with a cotton cloth lightly moistened with 70% alcohol.
- Do not clean the device with corrosive detergent.
- Keep the device away from water or other liquid.
- Store the device in a dry environment and keep it away from dust and direct sunlight.

14 Troubleshooting

If you happen to have one of the following problems while using the **Elma 609**, please refer to this breakdown service guide to help resolve the problem.

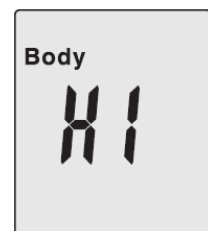
If the problem persists, please contact our customer service.

The LCD displays the body temperature less than 32°C (89.6°F)

If you're on **Surface mode** the 32°C (89.6 °F) temperature displayed is showing the external temperature that your body releases.

The LCD displays the message "HI"

When using the **Elma 609** the message "HI" can show on the screen. The analysis is above the measurement range selected, either superior to 42.5°C (108 °F) in **Body mode** or superior to 60°C (140°F) in **Surface mode**.

The screen displays the message "LO"

When using the **Elma 609** the message "LO" can show on the screen. The temperature analyzed is under the measuring range selected, either less than 32°C (90°F) in **Body mode** or less than 0°C (32°F) in **Surface mode**.

This message displays in various cases - please find below a list of the main cases.

Reasons for "LO" or "HI" message display	Advice
Temperature reading hampered by hair, perspiration. . .	Make sure that there is no obstruction prior to taking a temperature.
Temperature hampered by an air flux.	Make sure there is no air flux as this could interfere with the infrared system.
The measuring distance is too far.	Please respect the measuring distance (between 5 to 15cm - 2 to 5.9 in).

From high/low temperature condition to room temperature

Waiting for 10 minutes before taking the body temperature.

15 This appliance conforms to the following standards:

- EN12470-5 and ASTM E1965-1998
- EN 980: Graphical symbols for use in the labeling of medical devices
- EN 1041: Information supplied by the manufacturer with medical devices
- EN 60601-1: Medical electrical equipment Part 1: General requirements for safety (IEC:60601-1 :1998)
- EN 60601-1-2: Medical electrical equipment Part 1-2: General requirements for safety Collateral standard Electromagnetic compatibility Requirements and test (IEC 60601-1-2:2001)

EMC Statement

This device has been tested and homologated in accordance with EN60601-1-2:2007 for EMC. This does not guarantee in any way that the device will not be affected by electromagnetic interference. Avoid using the device in high electromagnetic environment.

The MEDICAL DELECTRICAL EQUIPMENT needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENTS.

The manufacturer reserves the right to alter the specifications of the product without prior notification.

The manufacturer allows himself the right to modify without any preliminary opinion the technical specifications of the product.

ELMA 609

Producer: Shenzhen Everbest Machinery Industry Co., Ltd.
19th Building, 5th Region, Baiwangxin Industrial Park, Songbai RD., Baimang, Xili,
Nanshan, Shenzhen, China

Importer:

Elma Instruments A/S
Ryttermarken 2
DK-3520 Farum
T: +45 7022 1000
F: +45 7022 1001
info@elma.dk
www.elma.dk

Elma Instruments AS
Garver Ytteborgsvei 83
N-0977 Oslo
T: +47 22 10 42 70
F: +47 22 21 62 00
firma@elma-instruments.no
www.elma-instruments.no

Elma Instruments AB
Pepparvägen 27
S-123 56 Farsta
T: +46 (0)8-447 57 70
F: +46 (0)8-447 57 79
info@elma-instruments.se
www.elma-instruments.se

Lotus NL B.V
Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.
E-mail: peter@lotusnl.com Tel : +31644168999